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**DIRECTION FOR USE**
**1. Usage area**

Thermotop is electrically heated device, electrically placing, without level regulated temperature for manufacturing of cap with dipping method in dental laboratory. With suitable wax you can get very good and precise results.

**2. Advantages**

Simple use, efficient, with electronically regulated temperature, always prepared to be used.

**3. Technical data**

Connection	230V,50/60Hz
Power taken away at max. temperature:	heating phase 85W, working 12W
Temperature range:	60 - 105°C without level regulation
Function signalization:	lightened switcher
Outside measurements:	width=82 mm, depth=80 mm, height=95 mm
Weight:	0,5 kg
Cable with protective connector:	1,5 m

**4. Description**

REF 0601 Thermotop is compact, efficient device for cap manufacturing with dipping method. Without level regulated working temperature enable precise result of dipping method. Regulation of temperature in range from 60 - 105°C.

**4.1. Before first use**

When you take new device from the box make sure it is not damaged from the transport. If you notice any damage return the device immediately. Put the device on desired place and connected it with the plug on heating container. Make sure that the net voltage is in accordance with data on the board of device. If it is appropriate than connect the device on the net.

**4.2. Instruction**
**4.2.1. Direction**

Fill the pot with suitable wax. Switch the device on. Green light on the switcher shows that the device is ready. Regulate the desired temperature with swivel knob. **Be careful!** Touching the edge of the pot can cause burns.

**4.2.2. Mistakes removing**

If the pot overheats, call authorized service, that will find and remove the mistake.

**4.3. Maintenance**

Cleaning of the pot is very important. Maintain clean device. Repairing of device must always perform authorized service. Interdent is not responsible for mistakes made by unauthorized services. Guarantee for device is one year and does not valid in case of using the device in other purposes that are described in this directions for use and in case of intervention in device by unauthorized service.

**5. Safety precaution**

Danger of burns at direct contact with pot for wax melting. The intervention in device can be made by qualified person. Do not use device on people.

**6. Content of supply**

REF 0601 Thermotop pot for dipping 1 pcs

**7. Accessories**

/

**8. Warranty**

Our recommendations and instructions are based on our own experiences. Using of device in any other purposes as described above is made on own responsibility. Quality of device is tested and assured.

SI - 3000 CELJE · Opekarniška 26 · Tel.: +386 (0)3 425-62-00 · Fax: +386 (0)3 425-62-10  
E-mail: info@interdent.cc · http://www.interdent.cc

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